



Lohmann & Rauscher

Tribute[®] Wrap **NEW STYLES**

Nighttime, foam gradient compression garment.



Off-the-Shelf



Evening & Nighttime



Low Activity



Supports the continued maintenance of daytime volume reduction during the evening and at rest.

Tribute® Wrap

Promoting skin health¹ with proven foam and fabric technology.

Tribute Wrap is an off-the-shelf nighttime foam gradient compression garment for self-management during the evening, nighttime, and low activity.




Off-the-Shelf



Evening & Nighttime



Low Activity

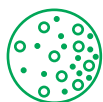

Proven foam technology

Engineered with first-use chipped foam technology to create therapeutic pressure differentials that facilitate a micro-massage effect.

- Designed to soften fibrotic tissue and break down hardened skin, supporting skin health¹.



POWERED BY  coolcore™



Powered by Coolcore™

Innovative fabric with chemical-free cooling technology

- 44% more wicking power³
- 17% more cooling power³
- 48% faster drying³
- Cooling solution that won't wash out for at least one year



Easy-to-pull finger grip

Simple pull function to support self-management. Finger holes reduce the need for a strong "pincher" grip.

- Can be used with button hooks/zipper pulls.

Tribute[®] Wrap

Nightly comfort meets effective results.

Why choose Tribute[®] Wrap?

Originally designed by therapists, Tribute Wrap offers a compression solution designed to empower users to take an active role in their daily self-care.

Its purpose is to prevent rebound swelling, maintain volume reduction and address soft tissue changes¹.

The adjustable straps help support self-care with its ease of application and conformability to patients' unique shape. Additionally, it promotes skin health with the differential pressures created from the chipped foam technology, which nestles into the tissue to promote a micro-massage-type effect.

Simply slide on, adjust the straps, and cover with the Sleep Sleeve* for comfortable, effective gradient compression throughout the night.


"This is outstanding! It is easier to put on my daytime flat knit garments, and my legs have less fatigue throughout the day.

I am convinced Tribute Wrap does make a significant difference!"

John, Lymphoedema patient



*Below Knee, Arm and Glove styles

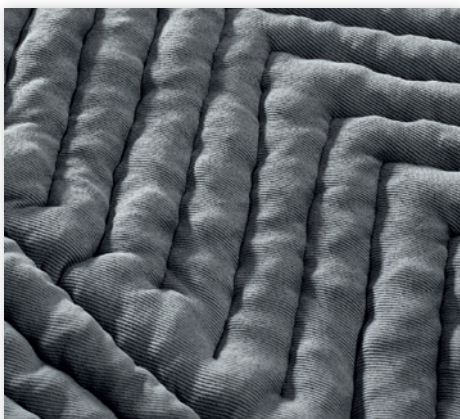

Chevron channelling

Chevron channelling based on manual lymphatic drainage (MLD) principles of directing lymphatic fluid to collateral pathways.

- Simulates the lymphatic drainage direction based on MLD
- Drains the lymph fluid to the lymph nodes from distal to proximal



Adjustable strapping system

Conforms effortlessly, ensuring a secure and comfortable fit that maintains therapeutic integrity.

- Design empowers individuals to fine tune as required



Supports mobility

Designed with contoured darting and angled straps at key flexion points, Tribute Wrap mirrors the body's natural movement to ensure a comfortable and functional fit.

- Reducing bulk at the joints removes physical barriers to movement, making it easier for users to independently manage their daily tasks and stay consistent with their therapy.


Tribute® Wrap Head & Neck NEW

Designed for easy self-care.

One size, two choices


Tribute Wrap Head & Neck gradient compression garments support easy self-care of head and neck-related lymphoedema and oedema during low activity by comfortably adjusting to your patients' facial and neck contour.

Tribute Wrap Head & Neck garments help maintain gains made during therapy, while allowing the flexibility to support further reduction. Simply adjust it around the neck and then on top of the head for comfortable home therapy!

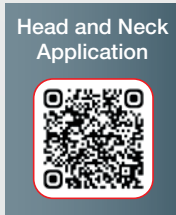

Head & Neck Head & Neck with Tracheostomy

sold individually, two 8 cm Compression Straps included

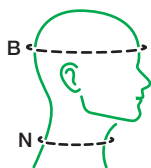

Style	Circumference (cm)		REF
	cB	cN	
Head & Neck	52-61	30-50	157396
Head & Neck with Tracheostomy			157397

Head & Neck garment styles do not utilise or include a Sleep Sleeve.

Head & Neck Accessories

Style	Size (cm)	REF
Pad (sold individually)		
Cheek Pad, Left	11.5 x 11.5	157628
Cheek Pad, Right	11.5 x 11.5	157629
Eye Patch Pad	9.5 x 8 x 3	157630
Bow Tie Pad	17.5 x 10	157631
Lip Bridge Pad	14.5 x 3.3	157632
Compression Strap		
8 cm Compression Strap (4 pcs)	8	157633
21 cm Crossed Compression Strap (2 pcs)	21	157634
28 cm Compression Strap (2 pcs)	28	157635

Optional Head & Neck Accessories

Accessories Application

All measurements in centimetres (cm).



Easy-to-don

Easy-to-use finger grips and clearly labelled straps. Only two VELCRO straps required for a comfortable fit.



Low profile fit

Designed to provide a slim and comfortable fit around the face and top of head.



Flexible ear comfort

Ears can be placed under strap or through opening, depending on patient preference. Supports hearing during garment use.




Focused foam

Foam around the neck and lower mandible to address fibrosis/hardened skin to promote skin health.



Chevron channeling

Designed to align with MLD practices, supporting lymphatic flow and the stimulation of superficial tissue.

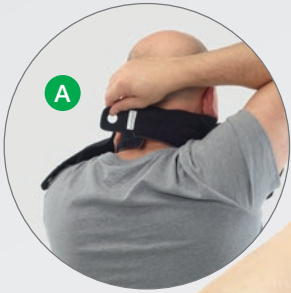

Adaptive to specific needs

Included compression straps are designed to provide a solution to address problematic areas of fibrosis and/or refine garment fit above or below tracheostomy or laryngectomy.

“My patient thought it was terrific and much more effective than the one he was using before (with a textured foam insert). He reported the garment was very comfortable and easy to use. He was able to adjust it to increase compression in specific areas and found this gave him excellent results.” Lymphoedema Therapist, NSW[^]


Tribute® Wrap

Solutions for upper and lower limbs



Glove


left or right orientation, sold individually, Sleep Sleeve included

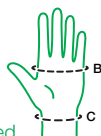

Size	Circumference (cm)		REF	
	Palm (cB)	Wrist (cC)		
S	16 – 20	16.5 – 21	Left	146783
			Right	146786
M	18 – 22	17.5 – 22	Left	146784
			Right	146787
L	20 – 24.5	18.5 – 23	Left	146785
			Right	146788

Wrist to Axilla

left or right orientation, sold individually, Sleep Sleeve included

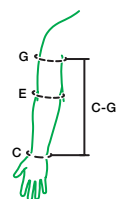

Size	Circumference (cm)			Length (cm)		REF
	Wrist (cC)	Elbow (cE)	Axilla (cG)	Arm (IC-G)		
S	16 – 21	22 – 28	25 – 31	43 – 47	Left	146777
					Right	146780
M	17 – 22	26 – 32	30 – 37		Left	146778
					Right	146781
L	18 – 23	30 – 36	35 – 44	Left	146779	
				Right	146782	

Sleep Sleeve


Complimentary black cover included with most styles to ensure the straps don't get tangled in the sheets.

- Below Knee Sleep Sleeve includes non-skid pads



Below Knee

left or right orientation, sold individually, Sleep Sleeve included

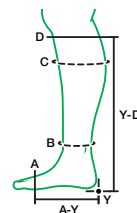

Size	Circumference (cm)		Length (cm)		REF	
	Ankle (cB)	Calf (cC)	Foot (IA-Y)	Calf (IY-D)		
S	24 – 30	30 – 36	19 – 22	40 – 44	Left	146789
					Right	146792
M	27 – 33	33 – 41.5			Left	146790
					Right	146793
L	30 – 36	36 – 45	Left	146791		
			Right	146794		

IA-Y is from the MTPs to heel.

Tribute® Wrap Bra NEW



Simply zip up and adjust!

Tribute Wrap Bra supports easy self-care of lymphoedema and oedema in the upper torso at nighttime or during low activity by comfortably adjusting to contour your patients' chest and axillary region. It helps maintain gains made during daytime therapy, while allowing the flexibility to support further reduction.

Simply zip it up and adjust the straps for the right fit in the comfort of your home!



“When I use the vest, I notice a significant difference in my comfort. It has made my post-activity recovery much more manageable, and I feel much more confident in my ability to independently manage my condition.”
Lymphoedema patient, NSW^

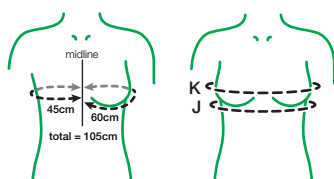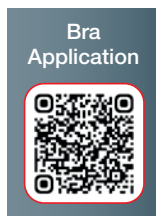

Bra
sold individually

Size	Circumference (cm)		REF
	cJ	cK	
Small	77-92	85-105	163667
Medium	87-102	95-115	163668
Large	97-112	105-125	163669
X-Large	107-122	115-135	163670
XX-Large	117-132	125-145	163671

Bra garment styles do not utilise or include a Sleep Sleeve.

- A** Bottom band adjustment strap accommodates a wide size range
- B** Downward-angled straps under bust provide contour and shaping around the lateral side of chest
- C** Upward-angled straps above bust provide contour and shaping around the lateral side of chest
- D** Shoulder straps stay aligned for easy positioning and comfort
- E** Long front zipper pull makes it easy to put on and take off
- F** Crew neck provides contoured non-restrictive neckline



Indications


- Mild to severe lymphoedema, stage 1–3
- Chronic oedema (non-blood related)

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Open wounds – any open wounds must be dressed prior to application

Contraindications

- Moderate to severe peripheral arterial disease
- Arterial insufficiency or ABPI < 0.8
- Untreated cancer
- Untreated congestive heart failure
- Untreated infection (e.g. Cellulitis) – allergies or sensitivities to materials
- Acute DVT
- Severe cognitive impairment

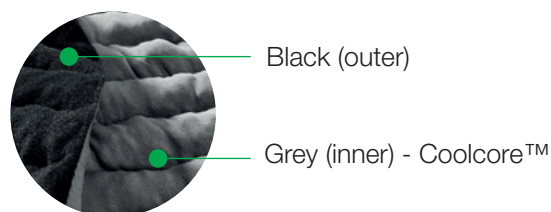

To Note

- 6-month warranty
- Easy garment care in washer (cold water) and dryer (low heat with other laundry items)
- Fasten straps prior to laundering
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls
- Not made with natural rubber latex

Product Composition

- Nylon (polyamide), polyurethane*, polyester, spandex (elastane)

*accessories with Pads only



References

1. Chohan, A., Haworth, L., Sumner, S., Olivier, M., Birdsall, D., and Whitaker, J. (2019). Examination of the effects of a new compression garment on skin tissue oxygenation in healthy volunteers. *Journal of Wound Care*. 28. 1.
2. Bjork R, Ehmann S. S.T.R.I.D.E. Professional guide to compression garment selection for the lower extremity. *Journal of Wound Care* 2019; 28:(6 suppl 1):1–44
3. Hohenstein Institute. Quantitative measurement of the evaporative heat loss of textiles (WATson Heat Loss Test): Coolcore® vs. Standard Performance Textiles. DIN SPEC 60015; 2019: 1–12

ReadyWrap® and Tribute® are trademarks of Lohmann & Rauscher GmbH & Co KG. © 2026

Lohmann & Rauscher Pty Ltd · 321 Ferntree Gully Road · Mount Waverley, Victoria 3149, Australia · Customer Service: 1300 572 869 www.lohmann-rauscher.com.au
 Lohmann & Rauscher Limited · 21 Queen Street · Auckland 1010 · New Zealand · Customer Service: 0800 572 869 www.lrcompression.co.nz



Nighttime therapy
with Tribute Wrap
lranz.com/tributewrap



Daytime therapy
with ReadyWrap®
lranz.com/ReadyWrapBrochure



To discover more about Tribute® Wrap and additional resources, visit
www.SupportforLife.com

Coolcore™ is a registered trademark of Coolcore, LLC.

VELCRO® is a registered trademark of VELCRO BVBA.

CertiPUR-US® is a certification program administered by a not-for-profit organisation. Although the CertiPUR-US® foam certification program is limited to and exclusively for flexible polyurethane foams used in bedding, pillows and upholstered furniture, our foams pass the same rigorous standards set by the CertiPUR-US® program.

^ Personal experience shared with permission. Individual results may vary. These testimonials reflect personal experiences and is not intended to represent clinical outcomes for every user. Always follow the specific compression garment application, indications, precautions and contraindications.